

Peroxidase Block

- Catalog No.:** K 033
- Intended Use:** This product is designed for use in peroxidase based immunohistochemical staining procedures done on paraffin tissue sections to suppress nonspecific staining due to endogenous peroxidase activity.
- Introduction:** Endogenous peroxidase activity has been observed in tissue sections during immunoperoxidase staining. To carry out immunoperoxidase staining, endogenous peroxidase must be suppressed from the tissue sections. If not properly suppressed, endogenous peroxidase can produce false immunostaining signals and create high background during immunostaining. Treatment of paraffin tissue sections with Peroxidase block prior to the application of primary antibody will suppress endogenous peroxidase activity.
- Format:** 50mL clear solution.
- Storage:** Light sensitive. Store at 2-8°C in dark bottle and/or area. Do not freeze. Do not use beyond the expiration date stated on the label.
- Procedure:** Incubate paraffin embedded tissue sections for 5-10 minutes at room temperature prior to applying primary antibody. Rinse tissue with wash buffer after incubation with peroxidase block and proceed to the next step.

IVD: For In Vitro Diagnostic Use

DBS will not be held responsible for patent infringement or other violation that may occur with the use of our product

DBS

1020 Serpentine Lane, # 114, Pleasanton, CA 94566 Tel: 925 484 3350, Fax: 925 484 3390

Website: www.dbiosys.com e-mail: customersupport@dbiosys.com