

Monoclonal Mouse Antibody to Human Lambda Light Chain

Catalog No.:	Mob 201, Mob 201-05
Intended Use:	This product is intended for qualitative immunohistochemistry with normal and neoplastic formalin-fixed, paraffin-embedded tissue sections, to be viewed by light microscopy. Clinical interpretation of staining results should be accompanied by histological studies with proper controls. Patients' clinical histories and other relevant diagnostic tests should be utilized by a qualified person(s) when evaluating and interpreting results.
Immunogen:	BALB/C mice were injected with purified human IgG myeloma proteins covalently coupled to polyaminostyren microbeads.
Clone:	HP6054
Isotype:	IgG2a
Format:	This antibody is supplied as purified immunoglobulin fraction containing sodium azide as a preservative.
Titer/Working Dilution:	This antibody may be diluted to a titer of 1:50-1:100 in an ABC method. The final dilution should be determined by the user based upon the staining conditions employed.
Staining Protocol:	We suggest an incubation period of 30 minutes at room temperature. Optimal incubation conditions should be determined by the user based upon the fixation conditions and staining system employed. <u>Formalin fixed paraffin embedded tissue sections require high temperature antigen unmasking with 10 mM citrate buffer, pH 6.0 prior to immunostaining.</u>
Specificity:	This antibody is specific to human lambda light chain. It reacts with free and bound heavy chain isotypes. Lambda light chains are found on the surface of B cells in lymphoid tissues. This antibody is useful for the identification of immunoglobulins in leukemias, plasmacytomas, and some non-Hodgkin's lymphomas.
Positive Control:	Tonsil
Cellular Localization:	Cytoplasmic
Storage:	Store at 2-8°C. Do not use beyond the expiration date stated on the label.
References:	i) Reimer et al. Hybridoma 3: 263, 1984.

IVD: For In Vitro Diagnostic Use

DBS will not be held responsible for patent infringement or other violation that may occur with the use of our product

DBS

1020 Serpentine Lane, # 114, Pleasanton, CA 94566 Tel: 925 484 3350, Fax: 925 484 3390

Website: www.dbiosys.com e-mail: customersupport@dbiosys.com