

Rabbit Monoclonal Antibody to Human Vimentin

Catalog No.:	RMAB 019
Intended Use:	This product is intended for qualitative immunohistochemistry with normal and neoplastic formalin-fixed, paraffin-embedded tissue sections, to be viewed by light microscopy. Clinical interpretation of staining results should be accompanied by histological studies with proper controls. Patients' clinical histories and other relevant diagnostic tests should be utilized by a qualified person(s) when evaluating and interpreting results.
Clone:	SP20
Immunogen:	Recombinant human vimentin protein.
Isotype:	IgG
Format:	This antibody is supplied as tissue culture supernatant containing sodium azide as a preservative.
Titer/Working Dilution:	This antibody may be diluted to a titer of 1:100 in an ABC method. The final dilution should be determined by the user based upon the staining conditions employed.
Staining Protocol:	We suggest an incubation period of 30 minutes at room temperature. Optimal incubation conditions should be determined by the user based upon the fixation conditions and staining system employed. No pretreatment is required for formalin fixed paraffin tissue sections.
Specificity:	This antibody recognizes vimentin, the main intermediate filament protein in mesenchymal cells and, thus, of value in the differential diagnosis of undifferentiated neoplasms. This antibody crossreacts with Human.
Positive Control:	Sarcomas
Cellular Localization:	Cytoplasmic
Storage:	Store at 2-8°C. Do not use beyond the expiration date stated on the label.

IVD: For In Vitro Diagnostic Use

DBS will not be held responsible for patent infringement or other violation that may occur with the use of our product

DBS

1020 Serpentine Lane, # 114, Pleasanton, CA 94566 Tel: 925 484 3350, Fax: 925 484 3390

Website: www.dbiosys.com e-mail: customersupport@dbiosys.com