

Rabbit Polyclonal Antibody to Human Lysozyme (Muramidase)

Catalog No.:	RP 028, RP 028-05
Intended Use:	This product is intended for qualitative immunohistochemistry with normal and neoplastic formalin-fixed, paraffin-embedded tissue sections, to be viewed by light microscopy. Clinical interpretation of staining results should be accompanied by histological studies with proper controls. Patients' clinical histories and other relevant diagnostic tests should be utilized by a qualified person (s) when evaluating and interpreting results.
Immunogen:	Lysozyme isolated from urine of patients with monocytic leukemia.
Host:	Rabbit
Format:	Purified immunoglobulin fraction of rabbit antiserum against human lysozyme containing sodium azide as a preservative.
Titer/Working Dilution:	This antibody may be diluted to a titer of 1:50-1:100 in an ABC method. The final dilution should be determined by the user based upon the staining conditions employed.
Staining Protocol:	We suggest an incubation period of 30 minutes at room temperature. Optimal incubation conditions should be determined by the user based upon the fixation conditions and staining system employed. <u>Suitable for formalin fixed, paraffin embedded tissue sections.</u>
Specificity:	This antibody reacts with lysozyme. It stains granulocytes, monocytes, and macrophages in human tonsil, skin, and colon. It does not cross-react with any other cell types.
Positive Control:	Tonsil
Cellular Localization:	Cytoplasmic
Storage:	Store at 2-8°C. Do not use beyond the expiration date stated on the label.
References:	i) Morsky et al. Clin Chim Acta 178: 327, 1988. ii) Krugliak et al. Am J Hematol 21: 99, 1986.

IVD: For In Vitro Diagnostic Use

DBS will not be held responsible for patent infringement or other violation that may occur with the use of our product

DBS

1020 Serpentine Lane, # 114, Pleasanton, CA 94566 Tel: 925 484 3350, Fax: 925 484 3390

Website: www.dbiosys.com e-mail: customersupport@dbiosys.com