

Rabbit Polyclonal Antibody to Human Thyroglobulin

Catalog No.:	RP 059, RP 059-05
Intended Use:	This product is intended for qualitative immunohistochemistry with normal and neoplastic formalin-fixed, paraffin-embedded tissue sections, to be viewed by light microscopy. Clinical interpretation of staining results should be accompanied by histological studies with proper controls. Patients' clinical histories and other relevant diagnostic tests should be utilized by a qualified person (s) when evaluating and interpreting results.
Immunogen:	Thyroglobulin isolated from human thyroid glands.
Host:	Rabbit
Format:	Purified immunoglobulin fraction of rabbit antiserum against human thyroglobulin containing sodium azide as a preservative.
Titer/Working Dilution:	This antibody may be diluted to a titer of 1:50-1:200 in an ABC method. The final dilution should be determined by the user based upon the staining conditions employed.
Staining Protocol:	We suggest an incubation period of 30 minutes at room temperature. Optimal incubation conditions should be determined by the user based upon the fixation conditions and staining system employed. <u>Suitable for formalin fixed, paraffin embedded tissue sections.</u>
Specificity:	This antibody reacts with human thyroglobulin containing cells in thyroid.
Positive Control:	Thyroid
Cellular Localization:	Cytoplasmic
Storage:	Store at 2-8°C. Do not use beyond the expiration date stated on the label.
References:	i) Burt et al. J Clin Pathol 38:893, 1985. ii) Lauberg et al. Scand J Clin Lab Invest 47: 685, 1987. iii) Kastrup et al. Scand J Clin Lab Invest 45: 471, 1985.

IVD: For In Vitro Diagnostic Use

DBS will not be held responsible for patent infringement or other violation that may occur with the use of our product

DBS

1020 Serpentine Lane, # 114, Pleasanton, CA 94566 Tel: 925 484 3350, Fax: 925 484 3390

Website: www.dbiosys.com e-mail: customersupport@dbiosys.com