

## Rabbit Polyclonal Antibody to Human Transferrin

<b>Catalog No.:</b>	RP 061, RP 061-05
<b>Intended Use:</b>	This product is intended for qualitative immunohistochemistry with normal and neoplastic formalin-fixed, paraffin-embedded tissue sections, to be viewed by light microscopy. Clinical interpretation of staining results should be accompanied by histological studies with proper controls. Patients' clinical histories and other relevant diagnostic tests should be utilized by a qualified person (s) when evaluating and interpreting results.
<b>Immunogen:</b>	Transferrin isolated from human plasma.
<b>Host:</b>	Rabbit
<b>Format:</b>	Purified immunoglobulin fraction of rabbit antiserum against human transferrin containing sodium azide as a preservative.
<b>Titer/Working Dilution:</b>	This antibody may be diluted to a titer of 1:50-1:100 in an ABC method. The final dilution should be determined by the user based upon the staining conditions employed.
<b>Staining Protocol:</b>	We suggest an incubation period of 30 minutes at room temperature. Optimal incubation conditions should be determined by the user based upon the fixation conditions and staining system employed. <u>Formalin fixed paraffin embedded tissue sections require high temperature antigen unmasking with 10 mM citrate buffer, pH 6.0 prior to immunostaining.</u>
<b>Specificity:</b>	This antibody reacts with human transferrin. It stains hepatocytes in fetal liver, epithelial cells of colon adenocarcinoma, and cells of normal liver. This antibody cross-reacts weakly with mink, raccoon dog and polecat transferrin-equivalent protein.
<b>Positive Control:</b>	Liver
<b>Cellular Localization:</b>	Cytoplasmic
<b>Storage:</b>	Store at 2-8°C. Do not use beyond the expiration date stated on the label.
<b>References:</b>	i) Faulk et al. Lancet ii: 390, 1980. ii) Mason et al. J Clin Pathol 31: 316, 1978. iii) May et al. J Membrane Biol 88: 205, 1985.

### IVD: For In Vitro Diagnostic Use

DBS will not be held responsible for patent infringement or other violation that may occur with the use of our product

**DBS**

1020 Serpentine Lane, # 114, Pleasanton, CA 94566 Tel: 925 484 3350, Fax: 925 484 3390

Website: [www.dbiosys.com](http://www.dbiosys.com) e-mail: [customersupport@dbiosys.com](mailto:customersupport@dbiosys.com)